

Labeling Blood Bank Specimens

1. Verify patient identification.
2. Place Blood Bank Identification (BBID) sticker horizontally on the tube



NOTE: DO NOT place BBID label upside down, 'TOP' must be aligned point toward the tube top/cap side



3. Place Cerner label below BBID label
4. Cover edge of the Blood Bank ID sticker




NOTE: If NOT performing PPID, DO NOT forget to manually write the collection date/time and Collectors ID



5. Wrap the label around the tube.
6. Do not cover the BBID.
7. Leave as much of the barcode exposed as possible



NOTE: Make sure label is on straight and not wrinkled

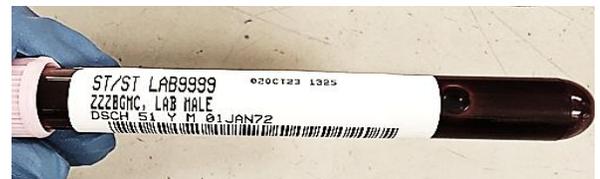

NOTE: Reference the Typenex Directions for Use for instructions on BBID wristband placement