


The 1, 2, 3's of Labeling Blood Specimens with PPID

1

Label Preparation

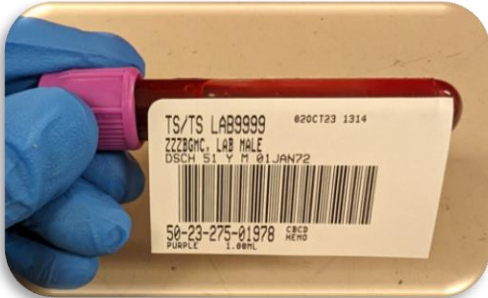

- Specimens collected **must** be labeled at the patient bedside
- Scan barcode on labeled specimen, once signed the collection information will be captured electronically.
 - Time of Collection
 - Date of Collection
 - Cerner ID

Note: Do Not Write *On* label

2

Label Placement



- Place the label lengthwise
 - Patient name by cap reading label from left to right
- Label should cover manufacturer label on tube
- Leave *window* to view blood volume

INCORRECT



Twisted labels, and labels placed too low on the tube cannot be read by the instrument



Properly labeled tubes result in faster TAT



CORRECT

