

JOB AID

INSTRUCTIONS FOR TYPENEX BLOOD BANK ARMBAND USING **MS4** LABEL


NOTE: IF PAID/PPID IS NOT USED, THE SYSTEM BLOOD BANK VERIFICATION FORM MUST ACCOMPANY THE SAMPLE TO THE LAB.

1. Obtain 2 pre-printed MS4 labels for the patient. Bring these labels and one Typenex Blood Bank Armband to the patient's bedside



2. Confirm patient's identity per facility defined protocol.
3. While at the patient's bedside:

- The MS4 labels should be legibly labeled with the collection date, time and Lan ID (network ID)
- If PAID/PPID is used no information should be written on the label, send Cerner labels with sample to lab.

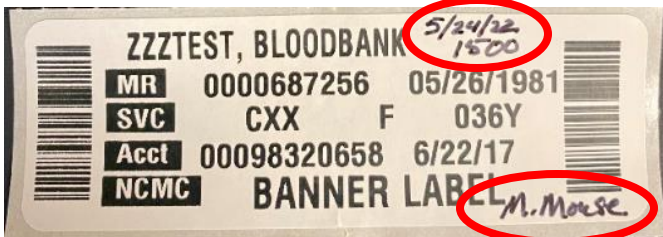

- Fold or trim the right-side barcode on the MS4 label so that it will fit into the "Patient info box" on the armband



- Peel the liner from the clear shield and seal over the patient MS4 label.



- Remove specimen tube sticker from Typenex band and place on sample tube.




4. Perform venipuncture per facility defined protocol.

5. Verify patient identity against printed MS4 labels to be certain of accuracy, then attach the Typenex band to the patient's wrist.

- Wrap band tail around patient's wrist to size.
- Snap the closure to secure and trim excess band with scissors.



6. Remove the tail of stickers from the band at the perforation

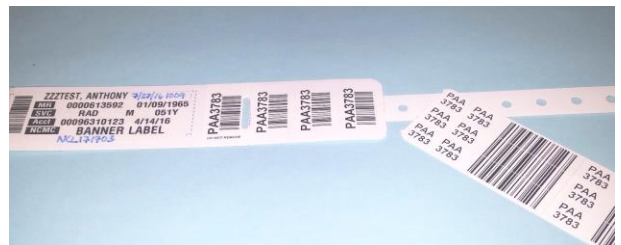

7. Apply second MS4 label onto the sample underneath the dashed line from the Typenex tube sticker.



Send all applicable Cerner Labels with the specimen to Lab.