




INSTRUCTIONS FOR CLINICAL STAFF
(see reverse for suspension instructions)

1. Check the expiry dates on Thinprep® vial and FLOQSwabs® (552C.RM) to make sure the materials are not expired.
2. Ensure patient understands this is a vaginal self-collection test.
3. Place one patient identification label onto the FLOQSwabs® (552C.RM) and one label on the Thinprep® vial.
4. Provide patient with FLOQSwabs® (552C.ROM) and patient collection instructions.
DO NOT PROVIDE THE ENTIRE KIT TO PATIENT.

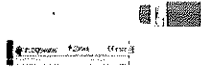

5. Direct the patient to a private room for collection (for example restroom, exam room)
6. After the patient returns and hands you the sample:
 - Remove Thinprep® vial and suspension instruction sheet from kit.
 - Follow instructions to suspend specimen in Thinprep® vial (see reverse for suspension instructions).
 - Complete order and ensure sample is properly labeled.
 - Place sample in the biohazard bag with absorbent sheet and place in storage area for pick-up by Quest courier.

Quest Diagnostics HPV Collection Pack (Rx Only)
Store the Collection Pack at room temperature
Questions? Call (866) MY-QUEST
Quest Diagnostics, Inc. Secaucus, NJ
Re-order no: K195

FOR QUEST INTERNAL USE ONLY – NOT FOR RE-SALE – NOT FOR HOME COLLECTION

Suspension of self-collected specimen


Sample suspension instructions for Copan FLOQSwabs® #552C.RM

Sample handling instructions for self-collected sample using Copan FLOQSwabs® #552C.RM for testing with the cobas® 4800 HPV Test or cobas® HPV.


Self-collected sample must be placed into medium after sample has been collected.

- Read all instructions before starting sample suspension.
- For sample collection, follow the collection device manufacturer's Instructions for use.
- Once the sample has been collected, continue with the following instructions to preserve the sample:


Handle the collected sample with care.

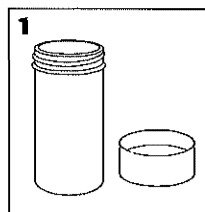


1. Carefully uncap the vial containing medium and place it on a stable, flat surface.

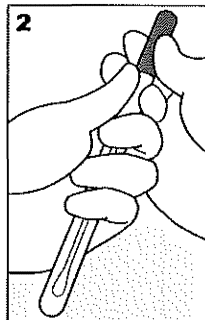


2. Slowly pull the FLOQSwab cap off to remove the swab from the tube. **Minimize touching the inner walls of the tube as you remove the FLOQSwab.**

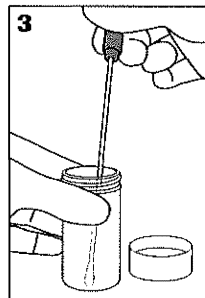


3. Hold the vial with one hand then with the other hand place the FLOQSwab tip into the vial until the FLOQSwab **tip is fully immersed in the medium and touching the bottom** of the vial.

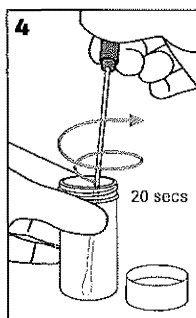

4. Holding onto the vial, swirl the FLOQSwab along the inner vial wall for 20 seconds while **ensuring the swab remains immersed** in the medium. Be careful not to splash.

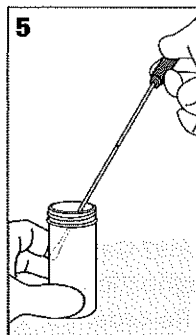

5. Carefully draw the FLOQSwab up along the inner vial wall until the tip is no longer immersed in the medium. Hold the tip against the inner vial wall to drain fluid off of the swab. Place the FLOQSwab into the tube and discard.

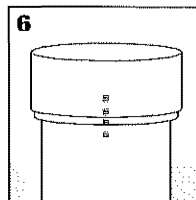


6. Re-cap the vial and tighten until the **lines on the cap and vial meet or slightly overlap** to prevent leakage. Store upright.

7. The sample can now be processed with the cobas® 4800 HPV Test or cobas® HPV.


Glossary



FLOQSwab/Swab: The self-collection device used to collect sample.

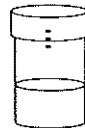

Vial: A container which contains 20 mL of clear solution. The specimen you collect will need to be transferred into this container and this container will be sent to the lab for processing.

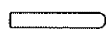

Tube: A protective container that the self-collected device will come in and can be used to temporarily store the collection device after the sample has been collected.

Medium: What the liquid that comes in the vial is called.