


# Panther Collection Devices and Test Association Guide

**Collection Device for: GONRNA, CHLRNA, TRICH RNA, VagPaBactRNA (BV), VagPaCTRNA (CV/TV), VagpaBCTCRNA (BV, CV/TV combo order), MYCO GENITAL (send out to Ruby)**

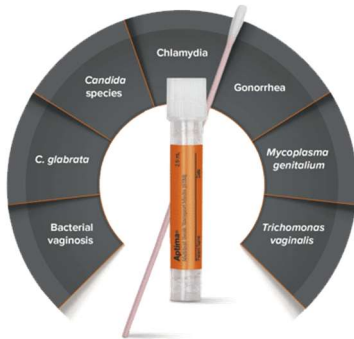

**Orange Multi Test swabs are HIGHLY VERSATILE.** Swab must be present in tube. If swab missing, send for recollect. Receive into LIS & place in the dirty area of the panther room or receiving bucket in accessioning area.

**Collection devices ONLY for: GONRNA, CHLRNA, TRICH RNA**



## Urine must be:

1. Dirty catch (i.e. cannot be used from UA specimen)
2. Less than 24 hours old and aliquoted within those 24 hours
3. Less than 30ml

Specimen should be **rejected** and sent for a **recollect** if it does not meet the criteria above.

Receive into LIS. Aliquot urine to the level of the window guide on the tube. Place into the dirty area in the panther room or receiving bucket in the accessioning area.



## Unisex swabs (not applicable for trich testing in males)

should have the blue swab visible in the tube to be received.

If it is white, wooden, or absent the specimen is rejected and sent for recollect.

Receive into LIS. Place into the dirty area in the panther room or receiving bucket in the accessioning area.

**Collection Device for GONRNA, CHLRNA, TRICH RNA & HPV**



Receive into LIS

Place into histology M-F 6am-4pm.

After hours, weekends, and holidays-

Place in mini fridge located in accessioning area.